beurer

Phototherapy unit TL 80

DE Tageslichtlampe Gebrauchsanweisung	2	FI Kirkasvalolamppu Käyttöohje	30
EN Daylight therapy lamp Instruction for use	9	DA Dagslys Lampe Brugsanvisning	37
FR Lampe de luminothérapie Mode d'emploi	16	NO Dagslys Lampe Bruksanvisning	44
SV Dagsljuslampe Bruksanvisning	23	RU Прибор дневного света Инструкция по применению	50

Inhalt

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Germany

Content

1. About the daylight therapy lamp	9	7. Replacing the lamp tubes/starter	13
2. Signs and symbols	10	8. Cleaning and care of the unit.....	13
3. Notes	10	9. Disposal	13
4. Unit description	11	10. Technical specifications	13
5. Initial use.....	12	11. Warranty/service.....	14
6. Operation	12		

1. About the daylight therapy lamp

Dear valued customer,

Thank you for choosing one of our products. Our name stands for high-quality, thoroughly tested products for applications in the areas of heat, weight, blood pressure, body temperature, pulse, gentle therapy, massage and air.

With kind regards
Your Beurer team

Why use a daylight therapy lamp?

When the hours of daylight are noticeably decreased in autumn and people increasingly stay inside, the effects of a lack of light may cumulate. This is often described as „winter depression“. The symptoms can present themselves in a number of ways:

- Imbalance
- Subdued mood
- Lack of energy and listlessness
- Generally feeling under the weather
- Need for more sleep
- Loss of appetite
- Difficulty concentrating

The cause of these symptoms is the fact that light, particularly sunlight, is essential for life and has a direct effect upon the human body. Sunlight indirectly controls the production of melatonin, which is released into the blood only in darkness. This hormone tells the body when it is time to sleep. In the months when there is less sunlight, therefore, production of melatonin increases. This makes it harder to get up in the morning, since the body's functions are shut down. If you use the daylight therapy lamp immediately after waking up in the morning—thus as early as possible—you can end the production of melatonin so that your mood can take a positive turn. In addition, light deficiency also hinders production of the happiness hormone serotonin, which decisively affects our well-being. So application of light involves quantitative changes in hormones and neurotransmitters in the brain, which can affect our level of activity, our feelings and our well-being. Light can help reverse this hormone imbalance. To compensate for such a hormonal imbalance, the daylight therapy lamp can create a suitable replacement for natural sunlight.

In the medical field, daylight therapy lamps are used to compensate for the effects of a lack of light. Daylight therapy lamps simulate daylight over 10,000 lux. The light can influence the human body and be used as a treatment or as a preventative measure. Normal electric light is not sufficient to influence the hormonal balance. Because in a well-lit office the illuminance level amounts to just 500 lux.

Included in delivery:

- Daylight therapy lamp
- These operating instructions

2. Signs and symbols

The following symbols appear in these instructions or on the device.

	Read the instructions of use		Permissible storage temperature and humidity
IP21	Protected against solid foreign objects 12.5 mm in diameter and larger, and against vertically falling drops of water		Permissible operating temperature and humidity
 Warning	Warning instruction indicating a risk of injury or damage to health		The CE labelling certifies that the product complies with the essential requirements of Directive 93/42/EEC on medical products
 Important	Safety note indicating possible damage to the unit/accessory	 Note	Note on important information
	OFF / ON		Manufacturer
	Serial number		Device in protection class 2
	Dispose of packaging in an environmentally friendly manner		

3. Notes

Please read these instructions for use carefully and keep them for later use, be sure to make them accessible to other users and observe the information they contain.

Safety notes

Warning

- The light of the daylight therapy lamp is only intended for use on the human body.
- Before use, ensure that all packaging materials are removed and that there is no visible damage to the unit or accessories. When in doubt, do not use the unit and contact your dealer or the customer service address provided.
- Also ensure that the daylight therapy lamp has a firm footing.
- The housing of the lamp gets warm during operation. Touching it may result in burns! Always let the lamp cool off before touching it.
- The mandatory minimum distance between the daylight therapy lamp and flammable objects is 1.0 m.
- Connect the unit only to the mains voltage listed on the type plate.
- Do not dip the unit into water and do not use it in wet rooms.
- Do not use it in the vicinity of people who are helpless, children under age 3 or people who are insensitive to heat (people with skin lesions due to illness); do not use after taking painkillers or drugs or drinking alcohol.
- Keep children away from packaging materials (risk of suffocation).
- Do not cover up or pack away the unit while it is warm.
- Always remove the mains plug and let the unit cool off before you touch it.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
- Protect the unit from strong impacts.

- Do not pull the mains plug from the socket by the cord.
- If the mains connection cable of this device is damaged, it must be disposed of. If it cannot be removed, the device must be disposed of.
- Disconnection from the mains supply is only guaranteed if the mains plug has been removed from the socket.
- To guard against the risk of electric shock, this device may only be connected to a power supply network equipped with a protective conductor.
- Keep the mains cable away from hot objects and naked flames.
- Do not make any changes to the device without the manufacturer's permission.
- If the device has been changed, thorough tests and checks must be carried out to ensure the continued safety of further use of the device.
- As the patient/user, you can operate all functions of this device.

General notes

⚠ Important

- If you are taking medication such as pain relievers, antihypertensives or antidepressants, consult your doctor before using the daylight therapy lamp.
- People with retinal diseases as well as diabetics need to be examined by an ophthalmologist before beginning to use the daylight therapy lamp.
- Please do not use if you have an eye disease such as cataracts, glaucoma, diseases of the optic nerve in general and inflammation of the vitreous humour.
- Always consult a doctor before using the daylight therapy lamp if you have a strong sensitivity to light or you are prone to migraine attacks.
- If you have health concerns of any kind, contact your general practitioner!
- Before you use the unit for the first time, remove all packaging materials.
- The lamp tubes are excluded from the warranty.

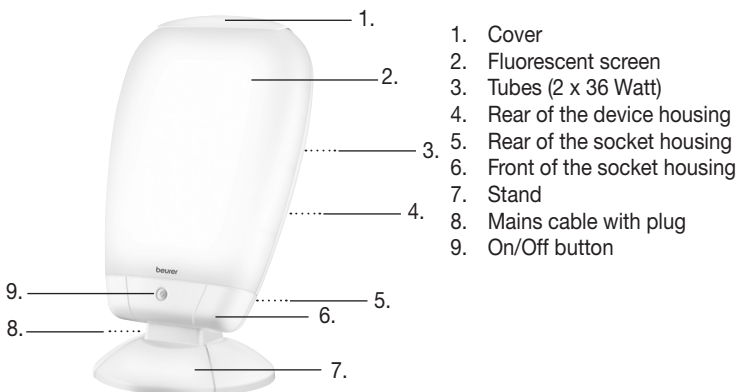
Repairs

⚠ Important

- Do not open the device. Please do not attempt to repair the unit yourself. This could result in serious injuries. Failure to observe this regulation shall void the warranty.
- If you need to have the unit repaired, contact customer service or an authorised dealer.

4. Unit description

Overview

5. Initial use

Setup

Take the device out of the plastic wrapping. Check the device for damage or faults. If you notice any damage or faults on the device, do not use it and contact customer services or your supplier. Place the device on flat surfaces. Select a location that leaves a distance of 20 to 50 cm between the unit and the user. The lamp is most effective within this range.

Mains connection

Connect the unit only to the mains voltage listed on the type plate. Push the plug completely into the mains socket.

Note:

- Ensure that there is a socket near the place where the lamp will stand.
- Lay the mains cable in such a way that no one can trip over it.

6. Operation

1	Switching on the lamp ▶ Press the On/Off button. The tubes turn on. Set the angle of the fluorescent screen in accordance with your individual requirements. ATTENTION! The brightness of the tubes may be visibly different when switched on. The tubes develop their full luminosity after a couple of minutes.												
2	Enjoying the light Sit as close as possible to the lamp, between 20 cm and 50 cm. You can go about your normal activities while using it. You can read, write, make telephone calls, etc. <ul style="list-style-type: none">• Every so often, briefly look directly into the light, since the effect results from the eyes/ retina receiving the light.• Use the daylight therapy lamp as often as you want. However, the treatment is at its most effective if you carry out the light therapy according to the prescribed time period of at least 7 successive days.• The most effective time of day for the treatment is between 6am and 8pm and we recommend that it is used for 2 hours per day.• Do not look directly into the light the whole time you are using it, however, since that could lead to overstimulation of the retina.• Start with shorter periods of illumination, which you increase over the course of a week. Note: Eye aches and headaches may arise after the first times using the lamp. These will go away in later sessions, as the nervous system will become accustomed to the new stimulation.												
3	Important instructions When using the lamp, maintain the recommended distance of 20 – 50 cm between your face and the lamp. The duration of the application depends on the distance: <table border="1" data-bbox="145 1260 873 1412"><thead><tr><th>Lux</th><th>Distance</th><th>Application duration</th></tr></thead><tbody><tr><td>10.000</td><td>approx. 20 cm</td><td>0,5 hour</td></tr><tr><td>5.000</td><td>approx. 35 cm</td><td>1 hour</td></tr><tr><td>2.500</td><td>approx. 50 cm</td><td>2 hours</td></tr></tbody></table> Basically, the closer you are to the source of light, the less time you should use it.	Lux	Distance	Application duration	10.000	approx. 20 cm	0,5 hour	5.000	approx. 35 cm	1 hour	2.500	approx. 50 cm	2 hours
Lux	Distance	Application duration											
10.000	approx. 20 cm	0,5 hour											
5.000	approx. 35 cm	1 hour											
2.500	approx. 50 cm	2 hours											

4	<p>Enjoying light over longer periods</p> <p>Repeat your use of the lamp in the dark seasons for at least 7 successive days, or even longer, depending on your individual needs. If possible, conduct the treatment in the morning hours.</p>
5	<p>Switching off the lamp</p> <p>▶ Press the On/Off button. The tubes turn off. Pull the mains plug from the socket.</p> <p>CAUTION!</p> <p>The lamp is hot after being used. Let the lamp cool off first long enough before you put it away and/or pack it up!</p>

7. Replacing the lamp tubes/starter

The service life of the fluorescent tubes is on average 10,000 hours. If one tube needs replacing, both fluorescent tubes must be replaced with new tubes of the same type. For this purpose, contact a retailer, an authorised service centre or appropriately qualified personnel or contact the customer service.

The following tubes are approved as replacement:

- YDW 36-H.RR 2G11
- The manufacturer's tubes are available with the order number 162.942 from customer support

Warning

- For environmental reasons, do not dispose of the fluorescent tubes with the household waste. Dispose of the unit at a suitable local collection or recycling point. Please contact the local authorities responsible for waste disposal.
- Unplug the device, to avoid the risk of electrocution.
- Take care when handling fluorescent tubes to avoid the risk of cut injuries.

8. Cleaning and care of the unit

The unit should be cleaned occasionally.

Important

- Ensure that no water penetrates inside the unit!
Before you clean the unit, always switch it off, unplug it and let it cool off.
- Do not wash the unit in a washing machine!
Clean using a damp cloth, to which you can apply a little detergent if necessary.
- Do not touch the unit with wet hands while it is plugged in; do not allow any water to be sprayed onto the unit. The unit must be operated only when it is completely dry.
Do not use abrasive detergents and never immerse the unit in water.

9. Disposal

For environmental reasons, do not dispose of the device in the household waste at the end of its useful life. Dispose of the device at a suitable local collection or recycling point. Dispose of the device in accordance with EC Directive – WEEE (Waste Electrical and Electronic Equipment). If you have any questions, please contact the local authorities responsible for waste disposal.

10. Technical specifications

Dimensions (WHD)	310 x 555 x 255 mm
Weight	2,7 kg
Lighting elements	YDW 36-H.RR 2G11
Power	72 watts
Light intensity	10,000 lux (Distance: approx. 20 cm)
Radiation	Output of radiation beyond the visible spectrum (infrared and UV) is so low that it is harmless to eyes and skin.

Product classification	Protection against moisture: IP21 Electrical protection class: class II
Mains connection	230 V ~, 50 Hz
Operating conditions	+5°C to 35°C, 30 – 90 % relative humidity; 700~1060hPa
Storage conditions	+5°C to 50°C, 30 – 90 % relative humidity; 700~1060hPa
Degree of efficiency of the ballast	~ 98%
Source efficiency (lm/W) of the bulb	64 lm/W
Color temperature of tubes	6.500 Kelvin

Subject to technical modifications.

The serial number is located on the device or in the battery compartment.

Brightness: 10,000 Lux (this light intensity value is just for information and regarding IEC 60601-2-57, this light source is classified as an exempt group).

Notes on electromagnetic compatibility

The device complies with the EU Medical Devices Directive 93/42/EEC, the German Medical Devices Act (Medizinproduktegesetz) and the European standard EN 60601-1-2 (in accordance with CISPR 11, IEC 61000-3-2, IEC 61000-3-3, IEC 61000-4-2, IEC 61000-4-3, IEC 61000-4-4, IEC 61000-4-5, IEC 61000-4-6, IEC 61000-4-8, IEC 61000-4-11) and is subject to particular precautions with regard to electromagnetic compatibility.

- The device is suitable for use in all environments listed in these instructions for use, including domestic environments.
- The use of the device may be limited in the presence of electromagnetic disturbances. This could result in issues such as error messages or the failure of the display/device.
- Avoid using this device directly next to other devices or stacked on top of other devices, as this could lead to faulty operation. If, however, it is necessary to use the device in the manner stated, this device as well as the other devices must be monitored to ensure they are working properly.
- The use of accessories other than those specified or provided by the manufacturer of this device can lead to an increase in electromagnetic emissions or a decrease in the device's electromagnetic immunity; this can result in faulty operation.
- Failure to comply with the above can impair the performance of the device.

11. Warranty/service

In case of a claim under the warranty please contact your local dealer or the local representation which is mentioned in the list "service international".

In case of returning the unit please add a copy of your receipt and a short report of the defect.

The following warranty terms shall apply:

1. The warranty period for BEURER products is either 3 years or- if longer- the country specific warranty period from date of Purchase.
In case of a warranty claim, the date of purchase has to be proven by means of the sales receipt or invoice.
2. Repairs (complete unit or parts of the unit) do not extend the warranty period.
3. The warranty shall not be valid for damages because of
 - a. improper treatment, e.g. nonobservance of the user instructions.
 - b. repairs or tampering by the customer or unauthorised third parties.
 - c. transport from the manufacturer to the consumer or during transport to the service centre.

- d. The warranty shall not be valid for accessories which are subject to normal wear and tear (cuff, batteries etc.).
4. Liability for direct or indirect consequential losses caused by the unit are excluded even if the damage to the unit is accepted as a warranty claim.

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Taishan Anson Electrical Appliances Co. Ltd • Dragon Mountain Industrial Estate
Distributed by BEURER GmbH • Söflinger Str. 218 • 89077 Ulm (Germany)
designed4inspiration GmbH • Feldgasse 34 • A-7072 Mörbisch am See